




bioLytical Laboratories INSTI® Multiplex HIV-1 / HIV-2 / Syphilis Antibody Test

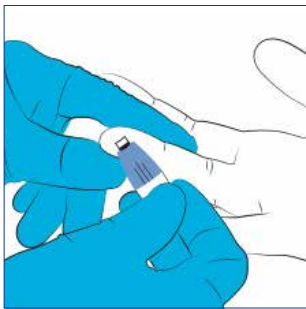


1 Clean fingertip with alcohol swab, allow to dry. Push finger gently. Place the single-use lancet firmly against the skin.

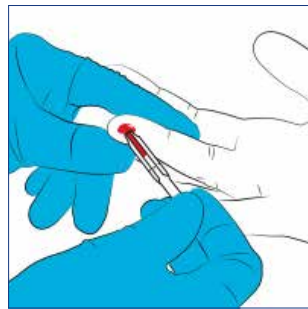

2 Squeeze the finger to allow sufficient blood flow. Place the single-use pipette (50 µl) horizontally into the blood bead. Allow the pipette to fill up until the black line. Do not squeeze the pipette.

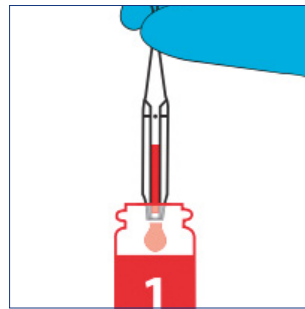

3 When the pipette is filled, transfer the blood into Solution 1 by squeezing the pipette bulb. Re-cap Solution 1 and gently invert the bottle a few times.

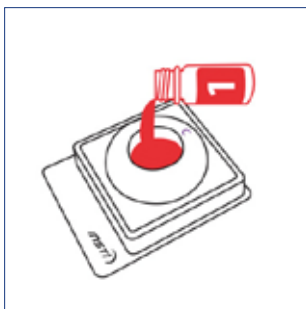

4 Open the pouch of the membrane unit without touching the test window. Immediately after, pour Solution 1 with the specimen into the center of the membrane unit test window. Solution 1 has to be poured into the test window within 5 minutes of adding the sample. Allow to absorb completely.



5 Mix Solution 2 thoroughly by slowly inverting the bottle. Pour Solution 2 into the center of the test window. Allow to absorb completely.



6 Pour Solution 3 into the center of the test window. Allow to absorb completely. Read the result immediately when Solution 3 has absorbed into the test window. Read the results latest within 5 minutes after the addition of Solution 3.

Please consult the instructions for use before performing the test.


bioLytical Laboratories INSTI® Multiplex HIV-1 / HIV-2 / Syphilis Antibody Test

Specimen collection and handling


- Before starting gather all material needed for testing: alcohol swab, single-use lancet, single-use pipette, unopened membrane unit and one vial of Solutions 1, 2 and 3.
- Test material: wholeblood, serum or plasma (for serum and plasma please refer to kit insert).
- Refer to the instructions for use to determine when the test controls should be run.
- Store at 15–30 °C.
- All test material should be used **immediately** once opened.
- All specimens should be handled as if capable of transmitting infectious diseases.
- Do not use reagents after expiry date.
- Do not mix components from different reagent lots.

Interpretation of results


Non-reactive/negative

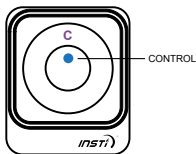

Indeterminate result

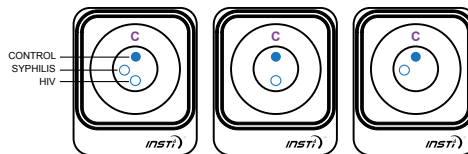


Syphilis & HIV reactive/positive

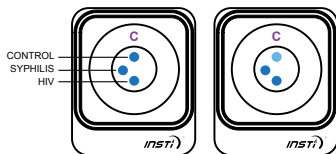


Invalid result



Syphilis reactive/positive

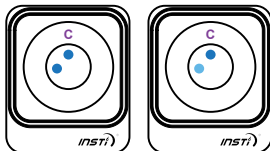

HIV reactive/positive

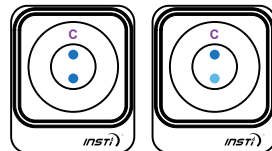

Contact and orders

Aidian Oy, Finland

aidian@aidian.eu / +358 10 309 3000
orders@aidian.eu

Product	Product no.	Order no.
INSTI® Multiplex HIV-1/HIV-2 Syphilis test, 1 pc	90-1028-002	155337
INSTI® HIV-1/HIV-2 Controls, 20 tests	90-1036	149874
INSTI® SYPH Control, 20 tests	90-1032	154850
INSTI® MICROSAFE® pipettes, 100 pc	1050-100	155111

AIDIAN

aidian.eu